

Test evaluation with patient sera

Sensitivity Data

Biognost® IFA Assays

Parameter:	ANAHEp
------------	--------

Parameter Description: Nuclear antibodies on Hep-2 cells

lower reference value:	0
number of sera tested:	35
standard deviation:	4776
medium titer:	3979
highest titer:	16384
number of sera < reference range:	0
sensitivity:	100%
data per:	11.09.2023

Patient sera are received:

- a) from renowned American blood banks, which supply us patients sera for positive controls;
- b) from institutions inside and outside Germany, which send in sera for double checking of their results;
- c) from Reference Laboratories (WHO, NIBSC, PEI etc.) inside and outside Germany, from which we purchase reference sera for test evaluation or which otherwise make characterized sera available to our enterprise.

All data listed have been obtained with single sera.